


Lansinoh®

Primo-Lacto® – Slip Tip

BPA
BPS
FREE

A Closed System for Colostrum Collection – Slip Tip Compatible

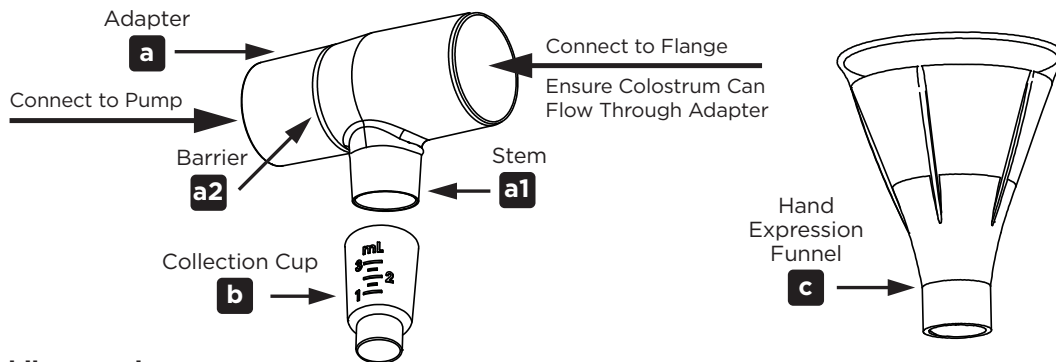

To use while pumping:

1. Wash hands with soap and water before each use.
2. Connect the adapter (**a**) to the breast pump flange. Press together to ensure a secure fit.
IMPORTANT: The adapter must be attached correctly for proper function. Colostrum must be able to flow from the flange and down the adapter stem (**a1**).
3. Connect the adapter (**a**) to the breast pump valve. Press together to ensure a secure fit.
IMPORTANT: The adapter must be attached correctly for proper function. The adapter barrier (**a2**) must be closest to the breast pump valve.
4. Press collection cup (**b**) onto the adapter stem (**a1**). Ensure a tight fit.
5. Attach bottle to breast pump valve per manufacturer instructions.
6. Press a slip tip compatible enteral feeding syringe into the bottom of the collection cup (**b**). Ensure a tight fit. Ensure syringe plunger is all the way up.
7. Begin pumping session. The colostrum will flow directly into the collection cup (**b**).
8. At the end of the pumping session, turn off pump and remove flange from breast. Pull the syringe plunger down to vacuum the expressed colostrum from collection cup (**b**) into the syringe. Remove the syringe. Cap and label the syringe. Follow your healthcare provider's instructions for feeding or storage.

PL59001IF0618

Primo-Lacto[®]

To use while hand expressing:

1. Wash hands with soap and water before each use.
2. Set the adapter **(a)** and collection cup **(b)** aside. You will only need the hand expression funnel **(c)**.
3. Press a slip tip compatible enteral feeding syringe into the bottom of the hand expression funnel **(c)**. Ensure a tight fit. Ensure syringe plunger is all the way up.
4. Begin hand expression and collect the expressed colostrum with the hand expression funnel **(c)**.
5. At the end of the session, pull the syringe plunger down to vacuum the expressed colostrum from the hand expression funnel **(c)** into the syringe. Remove the syringe. Cap and label the syringe. Follow your healthcare provider's instructions for feeding and storage.

Cleaning:

Wash with soap and hot water between sessions, unless otherwise instructed by a healthcare provider.

	Primo-Lacto [®]
Before 1st Use	✓ Sterile - no additional cleaning necessary
After Each Use	✓ Wash - use soap and hot water
Once Per Day*	✓ Sanitize - boil for 5 minutes

**For hospital use only during the colostrual stage of lactogenesis. One kit per patient.
Maximum usage is 3 days per kit. A second kit can be used as required.**

*Only necessary if directed by health care provider

© 2018 Lansinoh Laboratories
All Rights Reserved
Made in China
Distributed by
Lansinoh Laboratories, Inc.
Alexandria, VA 22314-4794
+ 1 800-292-4794